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**THANK YOU FOR PURCHASING  
THE INTICUP MENSTRUAL CUP!**  
Please read this manual thoroughly  
and follow the instructions.

### Easy to use

The Inticup menstrual cup is an easy to use, anatomically formed silicon cup worn inside the vagina to collect menstrual fluid. The package contains two cups: one for the lighter flow at the beginning and at the end of the menstruation as well as one for heavy flow. The Inticup offers you an easy period!

### Safe and durable

The Inticup is a safe, cost-efficient and eco-friendly option for pads and tampons. Since the menstrual cup does not change the fluid balance of the vagina or dry out the mucous membrane, it is safer than disposable feminine hygiene products. It also allows the vagina to naturally remove dead cells and bacteria along with the menstrual fluid. The Inticup is made from medical grade silicon, which is an easily cleaned and safe material. Besides being cost-efficient the menstrual cup is also eco-friendly, as it can be used again and again.

Using the Inticup is easy: Read the instructions on how to use and clean your menstrual cup to ensure several years of usage.

### INSTRUCTIONS:

Be patient when using the Inticup for the first time, as inserting the cup and using it might feel like a challenge at first. Don't worry, after a few periods you will notice how easy it is to use a menstrual cup!


### CARE INSTRUCTIONS – BEFORE AND AFTER EVERY PERIOD:

The menstrual cup is recommended to be washed with a mild soap before every use and after emptying. You can also rinse the cup with a fluid mixture of vinegar and water, which consists of nine parts water and one part vinegar.

**PLEASE NOTE:**  
*The cup should be sterilized in boiling water before using for the first time as well as after every period for a maximum of 3-5 minutes.*

### HOW TO INSERT THE MENSTRUAL CUP AFTER FOLLOWING THE CARE INSTRUCTIONS: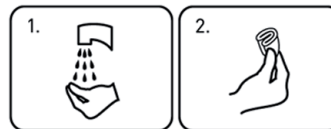


1. Always wash your hands before inserting the menstrual cup.
2. Press the cup into the shape of a C by placing your fingers on the marked area of the cup.
3. Sit down on the toilet or try another position with your legs apart for insertion. Insert the cup into your vagina while still holding the rim of the cup. Remember to stay relaxed and try not to tense your pelvic muscles.
4. Release your fingers from around the cup. Try feeling with your pelvic muscles if the cup is in place. By flexing your vaginal

muscles you will find the place most comfortable for you. Remember that the cup sits a bit lower in the vagina than a tampon, closer to the opening of the vagina. If the stem of the cup feels too long, you can cut it more suitable for you with scissors.

5. If experiencing difficulties inserting the cup, you can use water-based lubrication for insertion.

### REMOVING THE MENSTRUAL CUP:

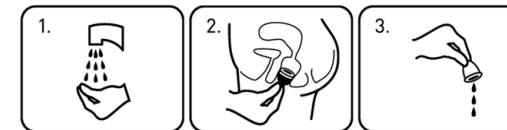1. Always wash your hands before removing the menstrual cup.
2. Sit down on the toilet or try another position with your legs apart for removing the cup. Squeeze the cup to remove the suction effect and pull the cup out from the vagina by the stem. Remember to stay relaxed and try not to tense your pelvic muscles. Support the cup with your fingers to avoid spillage.
3. Empty the cup and rinse it with some water and mild soap. Reinsert the cup. At the end of the cycle, remember to sterilize the cup with a mild disinfectant and put the cup in boiling water for five minutes.

NB: Empty the cup 2-6 times a day and every time you feel it is full during your cycle. Always keep your Inticup in the bag that came with the package, in a dry place away from sunlight. Always follow the instructions given.

CHECK OUT OUR WEB PAGE: [WWW.INTICUP.COM](http://WWW.INTICUP.COM)